888 Moo. 11 Bangsaotong, Muang Samutprakarn 10570

Tel. 02-397-9094-6, Fax 02-397-9097

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Polynet G-Tube is a Gastrostomy Silicone Tube for delivery of enteral nutrition and medication directly into the stomach. It is a silicone tube with a balloon on one side that is inserted into the abdomen leading to the stomach. The other side has 3 channels for delivering food, medicine and a check valve.

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Intended use : Gastrostomy Silicone Tube is intended used for oral feeding difficulties, but gastrointestinal function is normal, the need for long-term nutritional support, including Neurotic coma, anorexia, Alzheimer's disease. Mainly suitable for Gastrostomy patients.

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Indication : Gastrostomy Silicone Tube is indicated for use in patients who require long term feeding, who are unable to tolerate oral feeding, who are at low risk of aspiration and having gastric compression symptom and/or requiring medicine directly into the stomach.

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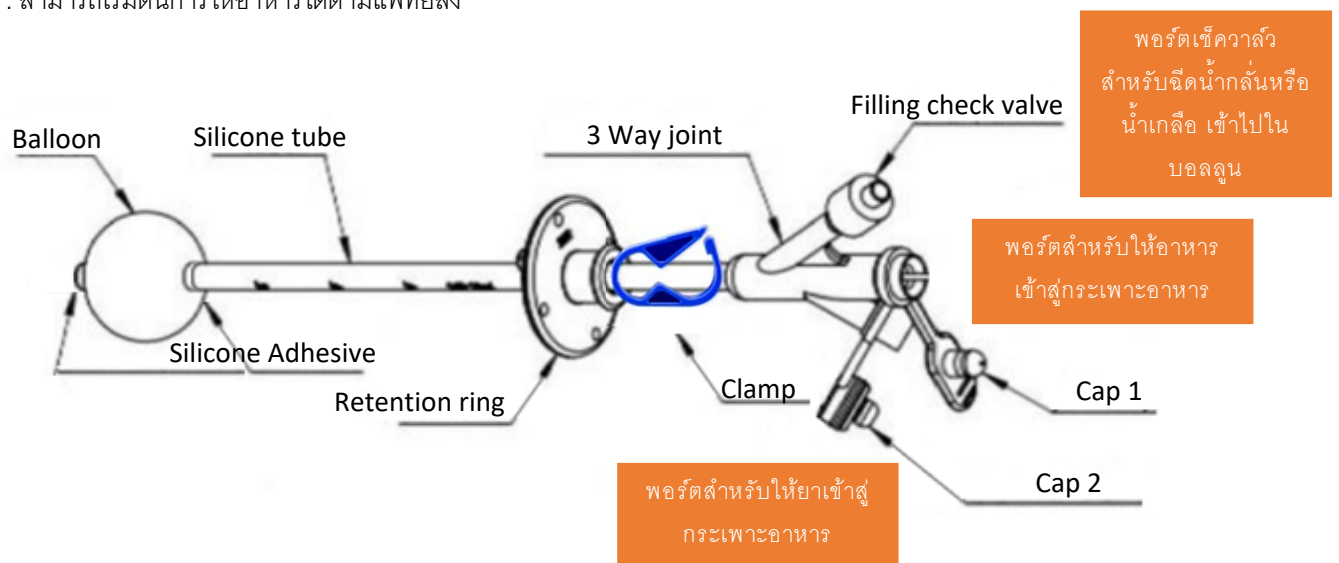
Insertion procedure for Gastrostomy Silicone Tube

1. Use lubricant to rub the end of the tube.
2. Put the patient in the supine position. Insert the end of the cannula slowly into the abdominal incision area. Push the tube into the stomach. Insert the balloon to fully insert into the stomach.
3. Use a syringe to inject sterile distilled water or saline solution into the balloon through the fill port with check valve according to the specified volume, approximately 10 – 20 cc.
4. Slowly lift the tube until the balloon touches the inside wall of the stomach.
5. Press down on the retaining ring. It shouldn't be too tight.
6. Check for leaks around the wound mouth. Check the position of the tube and the position of the retaining ring.
7. Can start feeding as ordered by the doctor.

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Postoperative and Feeding steps

1. After the Gastrostomy Silicone Tube is placed, the proportion and volume of feeding should be prescribed according to the prescription issued by the doctor.
2. The doctor prescribes an antibiotic regimen to prevent infection and evaluates the patient for any signs of infection, such as redness, irritation, edema, swelling, tenderness, fever, rash, suppuration.
3. Gastrostomy Silicone Tube feeding by provide ready-to-use milk type or mixed milk or liquid smoothies food recommended by a hospital nutritionist or doctor.
4. Wash your hands thoroughly and dry them. Then take a 50 cc syringe and connect it to the feeding hole at the edge of the tube at feeding port. Begin feeding by closing the feeding tube with the clamp to prevent old food from flowing back out. After that, pour the desired amount of liquid food into the cylinder by removing the inner core. Release the clamp. The food will flow into the stomach.
5. Flush the Gastrostomy Silicone tube with water during each feeding interruption or at least every 8 hours during continuous feeding.
6. Keep the skin around the wound clean, dry and free of leakage. The wound and surrounding skin should be inspected before each feeding, and the area should be washed with a mild soapy water solution at least once a day, making sure to clean under the retention ring, and be sure to rinse and dry the skin thoroughly.
7. Please change the new Gastrostomy silicone tube if require long term feeding more than 30 days.

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Removal of the Gastrostomy Silicone tube

1. Rotate the tube 360 degrees to make sure the tube moves freely and effortlessly.
2. Insert the syringe firmly into the balloon filling joint (Check valve) and withdraw all fluid.
3. Loosely cover the base tube area with a towel or surgical drape. Then hold the Gastrostomy Silicone tube straight, pull the tube with a steady pull force until the inner dome emerges from the abdominal wall.

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Warning

1. Please read the instruction before use. Use strictly in accordance with the instruction.
2. Inserting a Gastrostomy silicone tube into the abdomen, It must be performed and under the supervision of a skilled doctor only.
3. Do not use after expiration. See the product packaging for the production date and expiration date.
4. Before use, carefully check whether the packaging is in good condition, and it is forbidden to use if the original packaging is damaged.
5. This product is for one-time use and cannot be reused.
6. Be sure to use a Gastrostomy silicone tube that matches the size of the fistula.
7. During the indwelling process, be careful not to cause excessive compression of the abdominal wall and stomach wall.
8. Be sure to confirm that the edge of Gastrostomy silicone tube is indwelling in the stomach before starting nutrition.

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Contraindication

Contraindications for placement of a Gastrostomy silicone tube with the pregnant woman, colonic
interposition, hypertension, gastritis and obesity.

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Precaution

1. Reuse is prohibited.
2. If the fistula is not actually formed, please do not use this product when the fistula is damaged or abnormal.
3. Use sterilized distilled water for balloon expansion (if normal saline or contrast agent is used, the balloon
may not be deflated due to the coagulation of the components).
4. Do not modify the Gastrostomy silicone tube or joint in any way.
5. Strictly follow the Instructions for Use and Patient Care Manual included in each device. The patient care
manual should, at a minimum, accompany the patient and be explained to all those involved in the care of the
patient. The manufacturer's instructions are not a substitute for a doctor's advice.

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